




INTRADERMAL INJECTION SIMULATOR LF01008U INSTRUCTION MANUAL





Internal Structure

The simulator features eight sites for practicing intracutaneous injections — four on the inner and four on the outer aspect of the forearm. If fluid is properly injected, a characteristic skin welt will form. The welt is removed by withdrawing the fluid after practice. Each site is reusable by dozens of students.

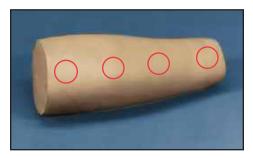


General Instructions for Use

It is recommended you use distilled water as an injection fluid with the simulator.

Procedures that Can be Performed on the Simulator

Intradermal or intracutaneous injections involve the injection of small amounts of material into the substance of the skin. This procedure is used for diagnostic procedures, allergy tests, and administration of regional anesthetics. The injections are given in an area where skin and hair are sparse, usually on the inner part of the forearm.

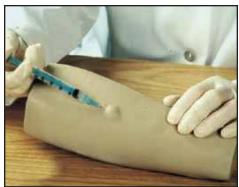

A 25-gauge needle, ³/8" to ¹/2" long, is generally used and inserted at a 10°-15° angle to the skin. **(See figure 1.)** Fluid is then injected to produce a small bleb just under the skin which causes a visible welt on the outer skin surface. Remove fluid from the welt by inserting a needle without the syringe attached.

Care of the Simulator

- A. Puncturing the skin with needles forms small cuts which will eventually lead to deterioration. Should leakage occur, inject the supplied sealant fluid into the welt. Allow to set overnight before using that site again.
- B. Normal surface soil can be removed with a mild soap and luke-warm water. Use REN cleaner (W09919U) to remove stubborn stains. Simply apply the REN cleaner to the soiled area and wipe with a clean, soft paper towel. DO NOT clean the simulator with solvents or corrosive materials as they will cause damage.
- C. NEVER place the simulator on any kind of printed paper or plastic. These materials, as well as ballpoint pens, will transfer indelible stains.

Supplies/Replacement Parts LF01009U 2-oz. Sealant

W09919U REN Cleaner

Figure 1

Other Available *Life/form*. Simulators

LF00698U Adult Injectable Arm (White) LF00855U Male Catheterization LF00856U Female Catheterization LF00901U Prostate Examination LF00906U Ostomy Care LF00929U Surgical Bandaging LF00957U Enema Administration LF00958U Pediatric Injectable Arm LF00961U Intramuscular Injection LF00984U Breast Examination LF00995U Arterial Puncture Arm LF00999U Pediatric Injectable Head LF01005U First Aid Arm LF01008U Intradermal Injection Arm LF01012U Heart Catheterization (TPN) LF01019U Ear Examination LF01027U Peritoneal Dialysis LF01028U Suture Practice Arm LF01034U Suture Practice Leg LF01036U Spinal Injection LF01037U Hemodialysis Practice Arm LF01038U Episiotomy Suturing Set LF01042U Suture Kit LF01062U Pelvic, Normal & Abnormal LF01063U Stump Bandaging, Upper LF01064U Stump Bandaging, Lower LF01069U Cervical Effacement LF01070U Birthing Station LF01082U Cricothyrotomy LF01083U Tracheostomy Care LF01084U Sigmoidoscopic Examination LF01087U Central Venous Cannulation LF01095U Blood Pressure Arm LF01108U Infant Intraosseous Infusion LF01121U Advanced IV Arm **LF01131U** Venipuncture and Injection Arm LF01139U Advanced IV Hand LF01142U Auscultation Trainer LF01143U Testicular Exam LF01152U Male & Female Catheter LF01155U Advanced CPR Dog LF01162U Venatech IV Trainer LF01174U NG Tube & Trach Skills

10101015	
LF01184U	Venatech IM & Sub Q
LF01193U	Special Needs Baby
LF03000U	CPARLENE [®] Series
LF03601U	Adult Airway Management
	Trainer with Stand
LF03602U	Adult Airway Management
	Manikin
LF03609U	Child Airway Management
	Trainer with Stand
LF03616U	Child CRiSis™ Manikin
LF03617U	Deluxe Child CRiSis™
	Manikin with Arrhythmia Tutor
LF03620U	PALS Update Kit
LF03623U	Infant Airway Management
	Trainer with stand
LF03632U	Child Intraosseous Infusion/
	Femoral Access Leg on a Stand
LF03633U	Child Airway Management
	Trainer Torso
LF03693U	Basic Buddy CPR Manikin
LF03699U	"Airway Larry" Airway
	Management Trainer
LF03709U	Infant CRiSis™ Manikin
LF03720U	Baby Buddy Infant CPR Manikin
LF03750U	Fat Old Fred
LF03760U	Airway Management/Cricoid
	Pressure Trainer
LF03770U	Chest Tube
LF03953U	CRiSis™ Manikin, complete
LF03955U	Deluxe CRiSis™ Manikin
LF03956U	Deluxe "Plus" CRiSis™ Manikin
LF03965U	Adult CRiSis™ Auscultation
	Manikin
LF03966U	Adult CRISIS™ Auscultation
	Manikin with ECG Simulator
LF04000U	GERi™/KERi™ Manikin Series
LF04200U	Adult Sternal Intraosseous
	Infusion
LF06001U	CPR Prompt™ Adult/Child
	Manikin
LF06012U	
LF06200U	CPR Prompt™ Keychain
	Rescue Aid
LF06204U	CPR Prompt™ Rescue and
	Practice Aid

901 Janesville Avenue, P.O. Box 901 Fort Atkinson, Wisconsin 53538-0901 1-800-558-9595 www.eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 1982 PRINTED IN U.S.A. NP 53-82/RV 8-07