


Nasco
Life/form®

**PERITONEAL DIALYSIS
SIMULATOR
LF01027U
INSTRUCTION MANUAL**

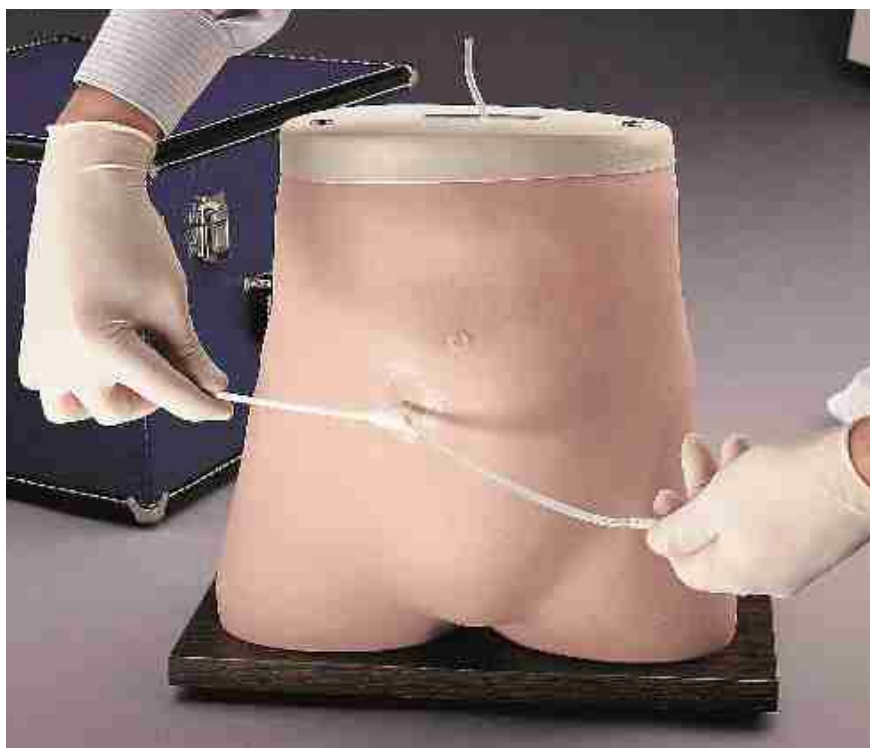

Life/form® Products by NASCO

About the Simulator...

This unique **Life/form**® Peritoneal Dialysis Simulator can provide the opportunity to practice and demonstrate all the procedures involved with peritoneal dialysis.

With a full capacity reservoir embedded inside the abdominal area, fluids can be introduced and emptied as they would be when treating a live patient.

Because extreme care and rigid procedures are so important in peritoneal dialysis, the opportunity provided by this simulator to get repeated and confirming practice can be very important.

Accurate, external anatomical detail is provided and ensured in this casting made from molds formed directly from a live volunteer.

NOTE: Original sponsorship for the development of this product was furnished by Abbott Labs.

Care and Cautions

1. Fluid Bags

The simulator uses an indwelling Tenckhoff catheter to allow the use of standard available fluid bags.

2. Use of Solutions

Do not use povidone-iodine prep solutions. These solutions will indelibly stain the simulator.

3. Cleaning

Most dirt and stains can easily be removed with a mild solution Ivory® liquid soap and water. Most stubborn stains can be removed with REN cleaner (W09919U)

4. Adhesive Bandages

Most kinds of adhesive bandages can easily be used. Use only Glue-Off (LF01023U) for removing residual adhesive. Apply thin coat. Allow to dry. Rub off in a circular motion.

5. Stains

The soft vinyl "skin" stains easily. Avoid contact with newsprint or other printed surfaces.

6. Storage

When not in continuous use, it is recommended to store the simulator in the storage case furnished, or cover with a cloth or sheet of plastic.

Supplies/Replacement Parts for Peritoneal Dialysis Simulator:

LF01023U

Glue-Off

LF09919U

REN Cleaner

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01037U Hemodialysis Arm
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01121U Advanced IV Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF03965U Deluxe "Plus" **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin
LF06001U **CPR Prompti**™ Adult/Child Manikin
LF06012U **CPR Prompti**™ Infant Manikin
LF06200U **CPR Prompti**™ Keychain Rescue Aid
LF06204U **CPR Prompti**™ Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com