



---

# HEMODIALYSIS PRACTICE ARM SKIN & VEIN REPLACEMENT KIT LF01040U INSTRUCTION MANUAL

---



**Life/form.** Products by NASCO

## List of Components:

- Replacement Skin
- Lace Tie
- Life/form®** Lubricant
- Vein Tubing

### 1. Remove Old Skin and Veins



Figure 1



Figure 2

- To remove the old skin from the arm, open the tie at the top end of the arm. Then, coat the exterior skin lightly with the lubricant (Figure 1) and strip off the skin so that it will be inside out when you finish (Figure 2).
- Remove the old vein tubing by pulling the tubing out of the channel in the arm core.

### 2. Place New Tubing in Channel

Double the tubing to find the approximate center, and place in the groove of the core starting with the center of the tubing at the center of the front of the wrist (Figure 3).

### 3. Lubricate Skin

- Pour the remaining lubricant into the skin allowing lubricant to run down the sides (Figure 4).

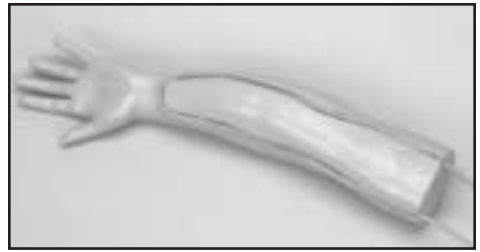


Figure 3



Figure 4



Figure 5

- Squeeze the skin between extended hands. Work hands back and forth, up and down the skin to coat the inside of the skin thoroughly (Figure 5).

### 4. Slide Skin On

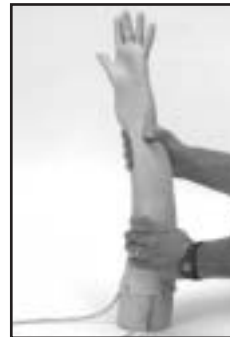


Figure 6

- Grasping the skin with both hands, slide the skin down over the core until the fingers are approaching the finger hole (Figure 6). **Caution:** Work the skin on gently, pulling hard on the end of the skin will stretch the material.

- B. As the skin is worked down over the core, the tubing may slip from the channel slightly. The tubing can usually be slipped back into position after the skin is slid completely onto the core.

## 5. Work Fingers into Place



**Figure 7**



**Figure 8**

Guide the fingers and thumb carefully into the proper holes as the skin is slipped further down on the core (Figure 7). If you encounter resistance, simply slide the skin back an inch or two from the fingers and begin again.

## 6. Draw Skin Tightly Over Core

- A. After the fingers are positioned in the hand, slide the skin over the remainder of the core to provide a snug fit (Figure 8).
- B. If air is trapped in the fingers, a 22 gauge needle can be used to puncture the finger tip and allow the air to escape.

## 7. Check Tubing Location

Feel along the tubing channels to be certain that the tubing is properly seated.

If it has slipped out of its channel, it can usually be moved back into position by working it with your fingers on the outside of the skin.



**Figure 9**

## 8. Position Ends of Tubing

Ends of the tubing should emerge from the center of the end of the core once the skin is tied (Figure 9).



**Figure 10**

## 9. Install Lace Tie and Rinse

- A. Thread the lace tie through the eyelets (Figure 10).
- B. Pull the skin snugly over the core and tie the lace to hold the skin in position. Check at this point to make sure that the tubing has not become kinked. Warm water should be used to rinse any excess lubricant remaining on the surface of the skin prior to use.
- C. If the skin stretches slightly during use, it can be tightened by retying the lace.

## Other Available *Life/form*® Simulators

<b>LF00698U</b>	Adult Injectable Arm (White)	<b>LF03610U</b>	Child Airway Management Trainer Head Only
<b>LF00856U</b>	Female Catheterization	<b>LF03611U</b>	Child Defibrillation Chest Skin
<b>LF00901U</b>	Prostate Examination	<b>LF03612U</b>	Child IV Arm
<b>LF00906U</b>	Ostomy Care	<b>LF03613U</b>	Child Blood Pressure Arm
<b>LF00929U</b>	Surgical Bandaging	<b>LF03614U</b>	Child Intraosseous Infusion/ Femoral Access Leg Only
<b>LF00957U</b>	Enema Administration	<b>LF03615U</b>	Complete Child <i>CRiSis</i> ™ Update Kit
<b>LF00958U</b>	Pediatric Injectable Arm	<b>LF03616U</b>	Child <i>CRiSis</i> ™ Manikin
<b>LF00961U</b>	Intramuscular Injection	<b>LF03617U</b>	Deluxe Child <i>CRiSis</i> ™ Manikin with Arrhythmia Tutor
<b>LF00984U</b>	Breast Examination	<b>LF03620U</b>	PALS Update Kit
<b>LF00995U</b>	Arterial Puncture Arm	<b>LF03621U</b>	Infant Airway Management Trainer Head Only
<b>LF00997U</b>	Adult Injectable Arm (Black)	<b>LF03622U</b>	Intraosseous Infusion Right Leg
<b>LF00999U</b>	Pediatric Injectable Head	<b>LF03623U</b>	Infant Airway Management Trainer
<b>LF01008U</b>	Intradermal Injection Arm	<b>LF03626U</b>	Child Femoral Access Injection Pad Replacement
<b>LF01012U</b>	Heart Catheterization (TPN)	<b>LF03627U</b>	Child <i>CRiSis</i> ™ Replacement Lung Set
<b>LF01019U</b>	Ear Examination	<b>LF03628U</b>	Child <i>CRiSis</i> ™ Replacement Stomach
<b>LF01020U</b>	Supplementary Ear Set	<b>LF03629U</b>	Child IV Arm Replacement Skin and Veins
<b>LF01025U</b>	Male Cath-Ed I	<b>LF03630U</b>	Child Arrhythmia Tutor
<b>LF01026U</b>	Female Cath-Ed II	<b>LF03631U</b>	Child Airway Management Trainer Replacement Headskin
<b>LF01027U</b>	Peritoneal Dialysis	<b>LF03632U</b>	Child Intraosseous Infusion/ Femoral Access Leg on a Stand
<b>LF01028U</b>	Suture Practice Arm	<b>LF03633U</b>	Child Airway Management Trainer with Torso
<b>LF01036U</b>	Spinal Injection	<b>LF03650U</b>	<i>CRiSis</i> ™ Manikin
<b>LF01053U</b>	Cross-Sectional Anatomy, Torso, Head	<b>LF03651U</b>	ALS Update Kit
<b>LF01054U</b>	Cross-Sectional Anatomy, Head	<b>LF03675U</b>	BLS Manikin — 10 Pack
<b>LF01062U</b>	Pelvic, Normal & Abnormal	<b>LF03676U</b>	BLS Manikin — 5 Pack
<b>LF01063U</b>	Stump Bandaging, Upper	<b>LF03677U</b>	BLS Manikin — Single
<b>LF01064U</b>	Stump Bandaging, Lower	<b>LF04001U</b>	<i>GERi</i> ™ Nursing Manikin
<b>LF01068U</b>	Brachial Plexus	<b>LF04020U</b>	<i>KERi</i> ™ Nursing Manikin
<b>LF01069U</b>	Cervical Effacement	<b>LF04021U</b>	<i>KERi</i> ™ Basic Manikin
<b>LF01070U</b>	Birth Station	<b>LF04022U</b>	<i>KERi</i> ™ Advanced Manikin
<b>LF01082U</b>	Cricothyrotomy	<b>LF04030U</b>	<i>GERi</i> ™ Advanced Manikin
<b>LF01083U</b>	Tracheostomy Care	<b>LF04040U</b>	<i>GERi</i> ™ Basic Manikin
<b>LF01084U</b>	Sigmoidoscopic Examination		
<b>LF01087U</b>	Central Venous Cannulation		
<b>LF01094U</b>	Cross Sectional Anatomy — Laminated		
<b>LF01095U</b>	Blood Pressure Arm		
<b>LF01108U</b>	Intraosseous Infusion Simulator		
<b>LF03000U</b>	<i>CPARLENE</i> ® Series		
<b>LF03601U</b>	Adult Airway Management Trainer		
<b>LF03602U</b>	Adult Airway Management on Manikin		
<b>LF03603U</b>	Adult Airway Management Head Only		
<b>LF03609U</b>	Child Airway Management Trainer		

*Nasco* Fort Atkinson

901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901  
(920) 563-2446

[www.eNASCO.com](http://www.eNASCO.com) • E-Mail [lifeform@eNASCO.com](mailto:lifeform@eNASCO.com)