



STUMP BANDAGING SIMULATORS INSTRUCTION MANUAL

FOR USE WITH UPPER STUMP (LF01063) & LOWER STUMP (LF01064)
BANDAGING SIMULATORS



Life/form® Products by NASCO

About the Simulator...

Featuring the realism of appearance always found with **Life/form**® products, these simulators consist of two separate parts to make the learning and demonstrating of stump bandaging much more effective and convenient for teacher and student. These simulators eliminate the need for live patients and give the students an opportunity to practice at their own pace.

List of Components

The upper torso includes the two arms, one amputated above the elbow and the other above the wrist. Both arms are slightly extended as a patient would have their arm while being bandaged. The compressibility of the material very closely duplicates that of a live patient and makes the experience very lifelike.

The lower half is designed in the shape of a prone patient with legs slightly abducted. One leg is amputated below the knee and the other above the ankle. The stumps are formed in the traditional way to add to the realism.

Procedures That Can Be Performed on These Simulators

Most standard bandaging procedures can be practiced or demonstrated. The lower torso is long enough to allow carrying the bandage around the body over the uninvolved hip at the level of the iliac crest. (Note: There is no embedded palpable skeletal material.)

Both units can be used to demonstrate the attachment of prosthetic devices.

Care of Simulator

Normal soil accumulated on the surface of the simulator can be removed with mild soap and lukewarm water. Use REN Cleaner (W09919U) to remove stubborn stains from simulator. Simply spray soiled area and wipe clean with a soft cloth or paper towels.

Most kinds of adhesive bandages can easily be removed. Use only Glue-Off (LF01023) for removing residual adhesives. Apply a thin coat. Allow to dry. Then rub off in a circular motion.

Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator on any kind of printed paper or plastic. These materials will transfer indelible stains. Ball-point pens will also make indelible stains.

Supplies/Replacement Parts for Stump Bandaging Simulators

LF01023U Glue-Off

W09919U REN Cleaner

Other Available *Life/form* Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01053U Cross-Sectional Anatomy, Torso, Head
LF01054U Cross-Sectional Anatomy, Head
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
LF03610U Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF04001U **GERI**™ Nursing Manikin
LF04020U **KERI**™ Nursing Manikin
LF04021U **KERI**™ Basic Manikin
LF04022U **KERI**™ Advanced Manikin
LF04030U **GERI**™ Advanced Manikin
LF04040U **GERI**™ Basic Manikin

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
 Fort Atkinson, Wisconsin 53538-0901
 1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com