




Heart Catheterization Simulator LF01012U Instruction Manual

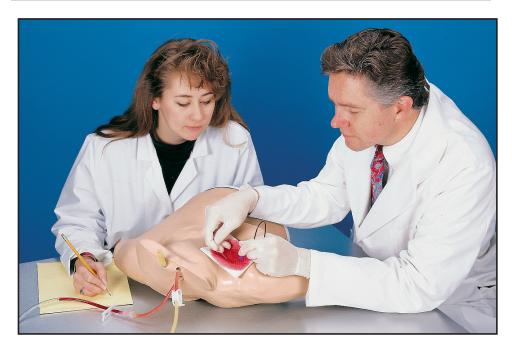

CAUTION: PRODUCT CONTAINS DRY NATURAL RUBBER. DO NOT REMOVE FILM FROM TUBING!



About the Simulator

The *Life/form*® Heart Catheterization Simulator is the ideal way to train staff in the techniques of central venous catheterization without patient risk. Venous catheterization and the use of a Swan–Ganz catheter can be practiced at all common sites of the upper torso (subclavian, jugular, and peripheral).

Preparation of the puncture sites and bandaging techniques may also be practiced with no fear of infection or harm to a patient.

Note: Some compromise in the position of the arm was necessary in the simulator to allow both peripheral and central catheterization. The right arm is slightly abducted, and the forearm is pronated. It is not necessarily the usual position for this procedure.

List of Components

- Replaceable Skin
- Vein Tubing Sealant Kit
- Needles
- Life/form® Venous Blood (1qt.)

Set Up Procedure

A. Prepare Synthetic Blood

Concentrated blood colorant is provided. Dilute the concentrate by filling the container with water.

B. Fill Supply Bag

Pour the colorant into the supply bag. Be certain the pinch clamp on the supply line is closed.

C. Connect to Simulator

Attach the connector on the end of the supply line to the matching connector on the simulator.

D. Fill System

Open the two pinch clamps on the simulator. Allow the system to be filled by opening the clamp on the supply line. Once the system is full, close both pinch clamps on the simulator. DO NOT close the pinch clamp on the supply line.

E. The simulator is now ready for use.

Skin and Tubing Replacement

Note how the tubing is laid out in the simulator while removing. This will make reassembling with new tubing easier.

- A. Remove clamps, connector, and shoestring.
- B. Lubricate exterior of outer skin with *Life/form*_® Lubricant Kit (LF00985U).


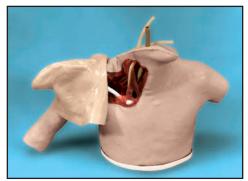


Figure 1

- C. Starting at the left shoulder, turn the skin inside out over the neck and chin area. (See figure 1.)
- Continue turning skin inside out down the entire length of the arm.
- E. Remove the white cover at the base of the simulator.



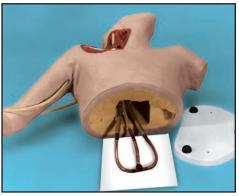

F. Pull the damaged tubes out the base of the simulator. (See figure 2.)



Figure 3

G. Lubricate the new tubing with Life/form® Lubricant Kit. (See figure 3.) DO NOT REMOVE FILM FROM TUBING. The new tubing can be inserted into the dissected area of the shoulder by attaching it to the wooden dowel. (See figure 4.)



Figure 4

- H. Position tubing in vein channels.
 - I. Lubricate the inside of new skin with *Life/form*_® Lubricant Kit (LF00985U).
- J. Slide new skin onto body.
- K. Connect clamps, connector, shoestring, and cover.
- L. The simulator is now reconditioned and ready for use.

Care of the Simulator

Store simulator in the accompanying carrying case. The simulator can be cleaned with Ivory[®] liquid soap and warm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains. Spray soiled area and wipe with soft cloth or paper towels.

Cautions

Solvents or corrosive materials will damage the simulator. Never place simulator on newsprint or any kind of printed paper or plastic. These materials will transfer an indelible stain. Ball-point pen ink will also make an indelible stain.

Supplies/Replacement Parts for the Heart Catheterization Simulator

LF00845U	<i>Life/form</i> _® Venous Blood (1 qt.)
LF01099U	Vein Tubing Sealant Kit
LF01013U	Replacement Skin and Vein Set
LF00985U	Life/form® Lubricant Kit
	Nasaa Claapar

LF09919U Nasco Cleaner

Other Available *Life/form*, Simulators

LF00698U Adult Injectable Arm (White) LF00855U Male Catheterization LF00856U Female Catheterization **LF00901U** Prostate Examination LF00906U Ostomy Care LF00929U Surgical Bandaging LF00957U Enema Administration LF00958U Pediatric Injectable Arm LF00961U Intramuscular Injection LF00984U Breast Examination LF00995U Arterial Puncture Arm LF00999U Pediatric Injectable Head LF01005U First Aid Arm LF01008U Intradermal Injection Arm LF01012U Heart Catheterization (TPN) **LF01019U** Ear Examination LF01027U Peritoneal Dialvsis LF01028U Suture Practice Arm LF01034U Suture Practice Leg LF01036U Spinal Injection LF01037U Hemodialvsis Practice Arm LF01038U Episiotomy Suturing Set LF01042U Suture Kit LF01062U Pelvic, Normal & Abnormal LF01063U Stump Bandaging, Upper LF01064U Stump Bandaging, Lower LF01069U Cervical Effacement LF01070U Birthing Station LF01082U Cricothyrotomy LF01083U Tracheostomy Care LF01084U Sigmoidoscopic Examination LF01087U Central Venous Cannulation LF01095U Blood Pressure Arm LF01108U Infant Intraosseous Infusion LF01121U Advanced IV Arm LF01131U Venipuncture and Injection Arm LF01139U Advanced IV Hand LF01142U Auscultation Trainer LF01143U Testicular Exam LF01152U Male & Female Catheter LF01155U Advanced CPR Dog LF01162U Venatech IV Trainer LF01174U NG Tube & Trach Skills

10101015	
LF01184U	Venatech IM & Sub Q
LF01193U	Special Needs Baby
LF03000U	CPARLENE [®] Series
LF03601U	
	Trainer with Stand
LF03602U	Adult Airway Management
	Manikin
LF03609U	Child Airway Management
	Trainer with Stand
LF03616U	Child CRiSis™ Manikin
LF03617U	Deluxe Child CRiSis ™
	Manikin with Arrhythmia Tutor
	PALS Update Kit
LF03623U	Infant Airway Management
	Trainer with Stand
LF03632U	
	Femoral Access Leg on a Stand
LF03633U	
	Trainer Torso
	Basic Buddy [®] CPR Manikin
LF03699U	
	Management Trainer
LF03709U	
LF03/20U	Baby Buddy™ Infant CPR Manikin
	Fat Old Fred
LF03760U	Airway Management/Cricoid Pressure Trainer
LF03770U	
	Crisis™ Manikin, Complete
	Deluxe <i>CRiSis</i> [™] Manikin
LF039550	
LF039560	
LF037030	Manikin
LF03966U	
EI 007000	Manikin with ECG Simulator
LF04000U	GERI™/KERI™ Manikin Series
	Adult Sternal Intraosseous
	Infusion
LF06001U	CPR Prompt [®] Adult/Child
	Manikin
LF06012U	CPR Prompt [®] Infant Manikin
	CPR Prompt [®] Keychain
	Rescue Aid
LF06204U	CPR Prompt [®] Rescue and
	Practice Aid
Fort Atking	son

Janesville Avenue, P.O. Box 901 Fort Atkinson, Wisconsin 53538-0901 1-800-558-9595 eNasco.com • E-mail: lifeform@eNasco.com COPYRIGHT © NASCO 1982 PRINTED IN U.S.A. NP 043-82/RV 5-12