

## Patient Education Tracheostomy Care Set LF01159U Instruction Manual



*Life*/*form*<sub>®</sub> Products by Nasco



#### About the Manikins

The Life/form® Patient Education Tracheostomy Care Set is ideal for teaching patients and caregivers the skills they will need to perform at home. The adult manikin includes oral and nasal passages, the pharynx, epiglottis, trachea, esophagus, tracheostomy site, and cricoid cartilage. The mouth and jaw are flexible to allow oral suctioning. The trachea branches into the left and right bronchial trees. Students can practice suctioning techniques, proper cuff inflation, dressing changes and other techniques. Cleansing and maintenance of the external tracheal site can be performed just as on an actual patient. The infant manikin can be used to teach patients and caregivers basic tracheostomy care skills.

#### List of Components

- Adult Tracheostomy Care Manikin
- Infant Tracheostomy Care Manikin
- Adult Tracheostomy Tube
- Infant Tracheostomy Tube
- Carrying/Storage Case
- Life/form<sub>®</sub> Lubricant

#### General Information About the Manikins

The manikins may be used in all methods of tracheostomy care. Individual methods and procedures may differ. Therefore, this booklet does not propose to outline a particular procedure to be followed. Rather, it is recommended that a procedural manual and instructor's directions be followed. This will eliminate confusion and best serve the individual instructor and curriculum.

## Procedures That May Be Performed on the Adult Manikin

- Oral-pharyngeal suctioning
- Nasal tracheal suctioning
- Tracheal suctioning via tracheostomy

#### Procedures That Can Be Performed on the Adult and Infant Manikin

- Dressing changes
- Stoma cleansing
- Changing of tracheostomy tube
- · Cuff inflation
- · Changing tracheostomy ties

#### Anatomy Present on the Adult Manikin

- Oral Passages
- Nasal Passages
- Pharynx
- Epiglottis
- Trachea
- Esophagus
- Stoma
- Right and Left Bronchi

#### Care of the Manikin

Normal surface soil can be removed with a mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains. Simply apply the Nasco Cleaner to the soiled area and wipe with a clean, soft paper towel.

#### Cautions

- Do not cleanse the stoma area with Betadine<sup>®</sup> or any other similar, colored disinfectant. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ball-point pens, will transfer indelible stains. Do not use any cosmetics on the simulator.
- With repeated use, the suctioning catheter will become soft and pliable, making it difficult to insert. You must either use a new catheter, or place the old one in a refrigerator or on ice for a few minutes to stiffen it.
- Heavy lubrication must be used while practicing procedures. The manikin will not function properly unless heavy lubrication is used. Use *Life/form*<sup>®</sup> Lubricant only. Do not use a water soluble lubricant.
- 4. Exercise the same care with this manikin as would be used with an actual patient. Excessive pressure and abuse will damage and shorten the unit's life.

# Supplies/Replacement Parts for the Patient Education Tracheostomy Care Set

- LF09919U Nasco Cleaner
- LF00985U Lubricant Kit
- LF01170U Adult Tracheostomy Tube
- LF01171U Infant Tracheostomy Tube

### Other Available *Life/form*. Simulators

LF00698U Adult Injectable Arm (White) LF00855U Male Catheterization LF00856U Female Catheterization LF00901U Prostate Examination LF00906U Ostomy Care LF00929U Surgical Bandaging LF00957U Enema Administration LF00958U Pediatric Injectable Arm LF00961U Intramuscular Injection LF00984U Breast Examination LF00995U Arterial Puncture Arm LF00999U Pediatric Injectable Head LF01005U First Aid Arm LF01008U Intradermal Injection Arm LF01012U Heart Catheterization (TPN) LF01019U Ear Examination LF01027U Peritoneal Dialysis LF01028U Suture Practice Arm LF01034U Suture Practice Leg LF01036U Spinal Injection LF01037U Hemodialysis Practice Arm LF01038U Episiotomy Suturing Set LF01042U Suture Kit LF01062U Pelvic, Normal & Abnormal LF01063U Stump Bandaging, Upper LF01064U Stump Bandaaina, Lower LF01069U Cervical Effacement LF01070U Birthing Station LF01082U Cricothyrotomy LF01083U Tracheostomy Care LF01084U Sigmoidoscopic Examination LF01087U Central Venous Cannulation LF01095U Blood Pressure Arm LF01108U Infant Intraosseous Infusion LF01121U Advanced IV Arm **LF01131U** Venipuncture and Injection Arm LF01139U Advanced IV Hand LF01142U Auscultation Trainer LF01143U Testicular Exam LF01152U Male & Female Catheter LF01155U Advanced CPR Doa LF01162U Venatech IV Trainer LF01174U NG Tube & Trach Skills

| Ulators    |  |
|------------|--|
| LF01184U   | Venatech IM & Sub Q  |
| LF01193U   | Special Needs Baby   |
| LF03000U   | CPARLENE <sup>®</sup> Series                                     |
| LF03601U   | Adult Airway Management  |
|            | Trainer with Stand   |
| LF03602U   | Adult Airway Management  |
|            | Manikin  |
| LF03609U   | Child Airway Management  |
|            | Trainer with Stand   |
| LF03616U   | Child <b>CRiSis™</b> Manikin                                     |
| LF03617U   |  |
|            | Manikin with Arrhythmia Tutor                                    |
| LF03620U   | PALS Update Kit  |
| LF03623U   | Infant Airway Management<br>Trainer with Stand                   |
| LF03632U   | Child Intraosseous Infusion/                                     |
| LF030320   | Femoral Access Leg on a Stand                                    |
| LF03633U   | Child Airway Management  |
|            | Trainer Torso  |
| LF03693U   | Basic Buddy <sup>®</sup> CPR Manikin                             |
| LF03699U   | "Airway Larry" Airway  |
|            | Management Trainer   |
| LF03709U   | Infant <b>CRiSis™</b> Manikin                                    |
| LF03720U   | Baby Buddy <sup>™</sup> Infant CPR Manikin                       |
| LF03750U   | Fat Old Fred   |
| LF03760U   | Airway Management/Cricoid  |
|            | Pressure Trainer   |
| LF03770U   | Chest Tube   |
| LF03953U   |  |
| LF03955U   |  |
| LF03956U   |  |
| LF03965U   |  |
|            | Manikin  |
| LF03966U   | Adult <b>CRiSis</b> ™ Auscultation<br>Manikin with ECG Simulator |
| 1 50/00011 | GERi <sup>™</sup> /KERi <sup>™</sup> Manikin Series              |
| LF04000U   |  |
| LF042000   | Infusion   |
| LF06001U   |  |
| El COUCHO  | Manikin  |
| LF06012U   |  |
| LF06200U   | CPR Prompt <sup>®</sup> Keychain                                 |
|            | Rescue Aid   |
| LF06204U   | CPR Prompt <sup>®</sup> Rescue and                               |
|            | Practice Aid   |
|            |  |

901 Janesville Avenue, P.O. Box 901 Fort Atkinson, Wisconsin 53538-0901 1-800-558-9595 eNasco.com • E-mail: lifeform@eNasco.com