



Patient Education Adult Tracheostomy Care Manikin LF01168U Instruction Manual



Life/form_® Products by Nasco



About the Manikin

The **Life/form** Patient Education Tracheostomy Care Adult Manikin is ideal for teaching patients and caregivers the skills they will need to perform at home. The adult manikin includes oral and nasal passages, the pharynx, epiglottis, trachea, esophagus, tracheostomy site, and cricoid cartilage. The mouth and jaw are flexible to allow oral suctioning. The trachea branches into the left and right bronchial trees. Students can practice suctioning techniques, proper cuff inflation, dressing changes, and other techniques. Cleansing and maintenance of the external tracheal site can be performed just as on an actual patient.

List of Components

- Adult Tracheostomy Care Manikin
- Adult Tracheostomy Tube
- · Carrying/Storage Case
- Life/form_® Lubricant

General Information About the Manikin

The manikin may be used in all methods of tracheostomy care. Individual methods and procedures may differ; therefore, this booklet does not propose to outline a particular procedure to be followed. Rather, it is recommended that a procedural manual and instructor's directions be followed. This will eliminate confusion and best serve the individual instructor and curriculum.

Procedures That May Be Performed on the Manikin

- · Oral-pharyngeal suctioning
- · Nasal tracheal suctioning
- Tracheal suctioning via tracheostomy
- Dressing changes
- · Stoma cleansing
- Changing of tracheostomy tube
- Cuff inflation
- · Changing tracheostomy ties

Anatomy Present on the Manikin

- Oral Passages
- Nasal Passages
- Pharynx
- Epiglottis
- Trachea
- Esophagus
- Stoma
- Right and Left Bronchi

Care of the Simulator

Normal surface soil can be removed with a mild soap and lukewarm water. Use Nasco Cleaner (LF09919U) to remove stubborn stains. Simply apply the Nasco Cleaner to the soiled area and wipe with a clean, soft paper towel.

Supplies/Replacement Parts for the Patient Education Adult Tracheostomy Care Manikin

LF09919U Nasco Cleaner

LF00985U Life/form_® Lubricant Kit

LF01170U Adult Tracheostomy

Tube

Cautions

- Do not cleanse the stoma area with Betadine® or any other similar, colored disinfectant. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ball-point pens, will transfer indelible stains. Do not use any cosmetics on the simulator.
- With repeated use, the suctioning catheter will become soft and pliable, making it difficult to insert. You must either use a new catheter, or place the old one in a refrigerator or on ice for a few minutes to stiffen it.
- Heavy lubrication must be used while practicing procedures. The manikin will not function properly unless heavy lubrication is used. Use Life/form_® Lubricant only. Do not use a water soluble lubricant.
- 4. Exercise the same care with this manikin as would be used with an actual patient. Excessive pressure and abuse will damage and shorten the unit's life.

Other Available *Life/form*. Simulators

Other Available <i>Life/form</i> , Simulators			
LF00698U	Adult Injectable Arm (Light)	LF01174U	NG Tube & Trach Skills
	Male Catheterization	LF01184U	Venatech IM & Sub Q
LF00856U	Female Catheterization	LF01193U	Special Needs Baby
	Prostate Examination		CPARLENE® Series
	Ostomy Care		Adult Airway Management
	Surgical Bandaging		Trainer with Stand
	Enema Administration	LF03602U	Adult Airway Management
	Pediatric Injectable Arm		Manikin
	Intramuscular Injection	LF03609U	Child Airway Management
	Breast Examination		Trainer with Stand
	Arterial Puncture Arm	LF03616U	Child CRiSis™ Manikin
	Pediatric Injectable Head	LF03617U	Deluxe Child <i>CRiSis</i> ™
	First Aid Arm		Manikin with Arrhythmia Tutor
	Intradermal Injection Arm	LF03620U	PALS Update Kit
	Heart Catheterization (TPN)	LF03623U	Infant Airway Management
	Ear Examination		Trainer with Stand
	Peritoneal Dialysis	LF03632U	Child Intraosseous Infusion/
	Suture Practice Arm		Femoral Access Leg on a Stand
	Suture Practice Leg	LF03633U	Child Airway Management
	Spinal Injection		Trainer Torso
	Hemodialysis Practice Arm		Basic Buddy® CPR Manikin
	Episiotomy Suturing Set	LF03699U	"Airway Larry" Airway
LF01042U			Management Trainer
	Pelvic, Normal & Abnormal		Infant CRISis™ Manikin
	Stump Bandaging, Upper		Baby Buddy™ Infant CPR Manikin
	Stump Bandaging, Lower		Bariatric CPR Manikin
	Cervical Effacement	LF03770U	
	Birthing Station		CRISis™ Manikin, Complete
	Cricothyrotomy		Deluxe CRISis™ Manikin
	Tracheostomy Care		Deluxe "Plus" CRISIs™ Manikin
	Sigmoidoscopic	LF03965U	Adult CRISIS™ Auscultation
E1010040	Examination	150007711	Manikin
LF01087U	Central Venous Cannulation	LF03966U	Adult CRISIs™ Auscultation Manikin with ECG Simulator
	Blood Pressure Arm	1 50400011	GERi™/KERi™ Manikin Series
	Infant Intraosseous Infusion		Adult Sternal Intraosseous
	Advanced IV Arm	LF042000	Infusion
-	Venipuncture and Injection Arm	1 E0600111	CPR Prompt® Adult/Child
	Advanced IV Hand	LFUOUUIU	Manikin
	Auscultation Trainer	LF06012U	CPR Prompt® Infant Manikin
	Testicular Exam		CPR Prompt® Keychain
	Male & Female Catheter		Rescue Aid
	Advanced CPR Dog	LF06204U	CPR Prompt® Rescue and
	Venatech IV Trainer		Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901 Fort Atkinson, Wisconsin 53538-0901 1.800.558.9595